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Operating Manual

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Lieferumfang des M-neb[®] vet

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ENGLISH

Scope of delivery M-neb[®] vet

4. The most important parts of the M-neb[®] vet
Please remove the M-neb[®] vet ultrasonic nebulizer, connection cable, nebulization unit and power cord from its packaging.

M-neb[®] vet
ultrasonic nebulizer
(Art.-No.: 20400200)

M-neb[®] vet
power cord
(Art.-No.: 20400202)

M-neb[®] vet
connection cable
(Art.-No.: 20400203)

M-neb[®] vet
nebulization unit
(Art.-No.: 20400201)


How to operate the M-neb[®] vet

5. Filling the M-neb[®] vet nebulization unit

Open the cap of the M-neb[®] vet nebulization unit and fill in the substance to be nebulized by means of a syringe (without hollow needle), an ampoule, etc. The maximum filling volume is 6 ml.

Please close the cap of the M-neb[®] vet nebulization unit afterwards.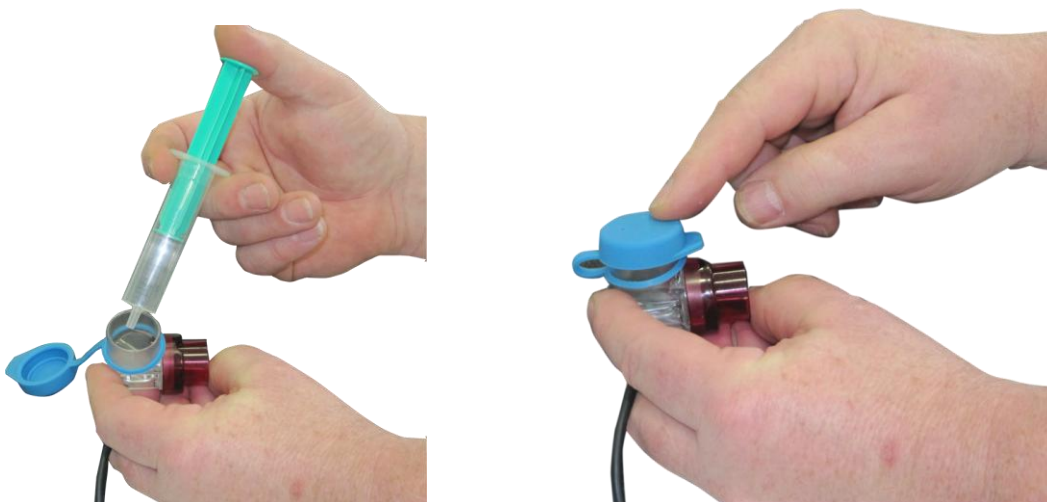

6. Connecting the M-neb[®] vet

Connect the enclosed connection cable with the M-neb[®] vet ultrasonic nebulizer and nebulization unit.

- d) For an inhalation treatment in dogs, connect the nebulization unit with the SaDoMa-II[®] inhalation mask and follow the operating manual starting page 16.
- e) For an inhalation treatment in cats, please follow the operating manual starting page 14.
- f) An inhalation treatment is also possible for all other small animals. Please contact the manufacturer for further information.

The M-neb[®] vet ultrasonic nebulizer can be used either with the M-neb[®] vet inhalation box (204 002 06) or the M-neb[®] vet inhalation mask (204 002 05).

Please follow these instructions while

- **using the M-neb[®] vet ultrasonic nebulizer together with the M-neb[®] vet inhalation box**
 1. Put your cat into the M-neb[®] vet inhalation box and close the cap. Please take care of the knobs which are inwrought in the cap and need to fit into the designated notches of the M-neb[®] vet inhalation box for an optimal closing.
 2. Afterwards please fill and connect the M-neb[®] vet nebulization unit according to the operating manual.
 3. Following this, please connect the M-neb[®] vet nebulization unit with the blue port of the M-neb[®] vet inhalation box cap.

Operating Manual of M-neb[®] vet ultrasonic nebulizer

4. It is recommended to cover the M-neb[®] vet inhalation box (for e.g. with a towel) so that as little aerosol as possible is exiting the box, but your cat still gets enough oxygen.
 5. For further usage please follow the operating manual of the M-neb[®] vet ultrasonic nebulizer.
- **using the M-neb[®] vet ultrasonic nebulizer together with the M-neb[®] vet inhalation mask**
 1. Fill and connect the M-neb[®] vet nebulization unit according to the operating manual.
 2. Afterwards please connect the M-neb[®] vet nebulization unit with the port of the M-neb[®] vet inhalation mask.

3. Sit down with your cat on your lap and lead the M-neb[®] vet inhalation mask to your cats' snout. Please pay attention that the cat inhales the produced aerosol properly.

4. For further usage please follow the operating manual of the M-neb[®] vet ultrasonic nebulizer.

Activation of inhalation

8. Activation of inhalation – intermittent mode

In order to activate the inhalation process press the on/off button for at least 1 second. The M-neb[®] vet ultrasonic nebulizer turns on automatically. You will hear a short acoustic signal and the capacity display is glowing green continuously. You have now entered the intermittent mode (1 second active / 1 second passive). The green LED is glowing continuously, the blue LED flashes alternately.

How to change the operating mode

9. How to switch to the continuous mode

Press the on/off button again shortly to switch to the continuous mode. A continuous nebulization takes place until the inhalant is completely nebulized. The green and blue LEDs are glowing permanently. You can return to the intermittent mode by pressing the on/off button again.

On/off button of the **M-neb[®] vet**
ultrasonic nebulizer

Completion of inhalation

10. Manual completion of inhalation

In order to pause or stop the inhalation process untimely, press the on/off button for at least 2 seconds. The M-neb[®] vet ultrasonic nebulizer turns off automatically which is accompanied by a short acoustic signal and the battery status bar will switch-off.

11. Automatic completion of inhalation

The automatic switch-off stops the M-neb[®] vet ultrasonic nebulizer 60 seconds after the filled medication is completely nebulized (independent of the set mode).

Charging the M-neb[®] vet ultrasonic nebulizer

12. Charging the M-neb[®] vet ultrasonic nebulizer

The M-neb[®] vet ultrasonic nebulizer needs to be charged once the battery status bar is flashing green and red alternately or is glowing red permanently. The battery of the M-neb[®] vet ultrasonic nebulizer may only be charged by using the corresponding power cord. For this purpose connect the power cord with the M-neb[®] vet ultrasonic nebulizer and the electric power supply. Disconnect the M-neb[®] vet ultrasonic nebulizer from the power cord and electric power supply afterwards.

Cleaning the M-neb[®] vet

13. Cleaning the M-neb[®] vet ultrasonic nebulizer and connection cable

Only wipe the housing of the M-neb[®] vet ultrasonic nebulizer and the connection cable by using a damp cloth. Dry both parts afterwards by using a dry cloth or let it air-dry by placing the parts on a dry and absorbent pad.

14. Cleaning the M-neb[®] vet nebulization unit

Open the blue cap of the M-neb[®] vet nebulization unit. Now you can rinse it under running tap water.

Important:

The connection cable with plug may not be in contact with water and therefore not be wet (risk of short circuit). Never pull directly on the cable as that may lead to cable breaks and equipment failures.

Note:

The nebulization units are separately available and are not covered by warranty. By careful handling and daily cleaning the operational life can be extended correspondingly.

Article list M-neb[®] vet

Article-No.:	Image:	Description:
20400001		M-neb [®] vet Starter Kit
20400200		M-neb [®] vet ultrasonic nebulizer
20400201		M-neb [®] vet nebulization unit
20400202		M-neb [®] vet power cord
20400203		M-neb [®] vet connection cable

Nebu-Tec International
med. Produkte Eike Kern GmbH
Kreuzfeldring 17
63820 Elsenfeld - GERMANY

Tel.: +49 (0) 6022 – 610-62-0
Fax: +49 (0) 6022 – 610-62-99

Email: info@nebu-tec.de
Web: www.nebu-tec.de